

HOW TO USE



**TYPE I
SHORT**



Place the flexible tip on the adjustment tooth and slide in sharpened tip to margin



Hold on the flexible tip with a "finger" for stability and press the handle gently



Temporary crown begins to lift up, confirming the loosening, remove the crown



**TYPE II
LONG**



Place the flexible tip on the adjustment tooth and slide in sharpened tip to margin



Hold on the flexible tip with a "finger" for stability and press the handle gently



Temporary crown begins to lift up, confirming the loosening, remove the crown



**TYPE III
SILICON**



Place the flexible tip on the adjustment place and slide in sharpened tip to margin



Press the handle gently



Temporary crown begins to lift up, confirming the loosening, remove the crown

Instruction Movie



TYPE I



TYPE II



TYPE III