



RELIANCE ORTHODONTICS INTRODUCES


# First Ever Orthodontic No Mix Self Etch Primer

ULTRA S.E.P. features a single liquid application – making effective tooth preparation quick, simple, and error-free.



- ✓ EFFECTIVE AND EFFICIENT TOOTH PREP
- ✓ REDUCES CHAIR TIME WITH SIMPLIFIED PROCESS
  - Scrub onto enamel for 5 seconds
  - Air dry
  - Apply Assure Plus (optional)
  - Air dry
  - Deliver bracket and cure
- ✓ SINGLE LIQUID APPLICATION NO MIXING
- ✓ GREAT FOR BOTH INITIAL BONDING AND EMERGENCY REPAIR APPOINTMENTS
- ✓ EFFECTIVE FOR ALL ENAMEL SURFACES
  - Bracket and Tube Bonding
  - Buildups
  - Increasing band cement retention
  - Attachment delivery

ONLY FROM RELIANCE®

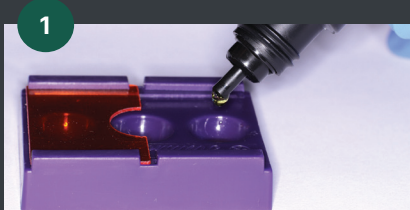


1 Dispense Ultra S.E.P. into chairside well, no mixing required

OR




1 Dispense Ultra S.E.P. into Reliance dual ring wells. No mixing required


OR

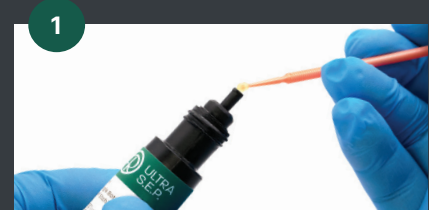

1 Drastically reduce material cost on REBONDS by saturating a microbrush with a single-tooth vertical dispense

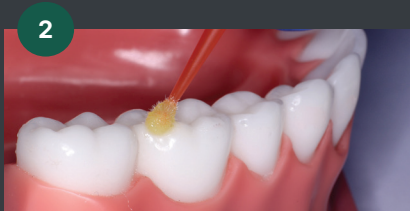

2 With a micro-brush, aggressively scrub Ultra S.E.P. onto tooth for 5 seconds

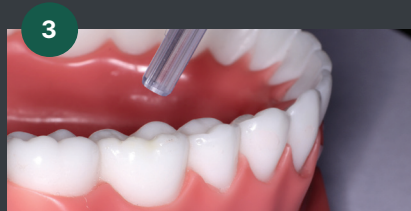

3 Air dry for 2-3 seconds before placing attachment/appliance



### ASSURE PLUS

Increase any S.E.P. strength with Assure Plus\*

\*Study protocols and results available at [RelianceU.tech](http://RelianceU.tech)



Order Hotline : (852) 2561-0012

(852)9148-4469

Fax : (852) 8148-7673

Email : [coopertc3805@gmail.com](mailto:coopertc3805@gmail.com)